



M760: Stretcher
User manual

Equipment M760

The M760 is an equipment with fixed stretcher and manually operated. Its main objective is the transport of patients with mobility difficulties and post-operation condition. This product is developed and produced by Auto Ribeiro Lda..

Equipment's Overview



Equipment's Lateral View



Equipment's Top View



SPECIFICATIONS

- Approximate Weight:
43 kg
- Maximum load allowed:
250kg
- Approximate Dimensions (closed):
1920 x 570 x 475 mm
- Approximate Dimensions (open):
M760 1920 x 570 x 1015 mm
M760S 1915 568 930 mm
M760H 1915 568 1090 mm

FRONT LEG CONTROL (Intermediate Position)

Security System



1° - Lift the lever (placed on the right side of the cart).



Final Position.

WHEELS

Swivel Wheels



2 Swivel Wheels

Fixed Wheels



2 Fixed Wheels



Locking system of the fixed wheels

1° Engage the red system with the foot to lock the wheels.

REAR LEG CONTROL

Security System



1° - Lift the safety lever.



2° - Press the **left** lever until it moves and keep it pressed..


(Note: To put the back leg into halfway, should press the lever and release it immediately)

FRONT LEG CONTROL

Security System



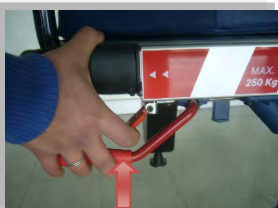

1° - Lift the safety lever.





2° - Press the **right** lever until it moves and keep it pressed..

TELESCOPIC HANDLES

<u>Security System</u>	
 A close-up photograph of a hand in a blue sleeve pressing a red button on a black telescopic handle. A red arrow points to the button.	1° - Press button with finger.
 A close-up photograph of a hand in a blue sleeve holding a black telescopic handle. A red arrow points horizontally, indicating the direction of push or pull.	2° - Push or pull to achieve the desired length.

SWING DOWN SIDEARMS

<u>Security System</u>	
 A photograph showing a hand in a blue sleeve pressing a red button on a sidearm of a stretcher. A red arrow points to the button. The sidearm is labeled 'ar equipment'.	1° - Press button with hand.
 A photograph showing a hand in a blue sleeve pushing or pulling the sidearm of a stretcher. A red arrow points horizontally, indicating the direction of movement. The sidearm is labeled 'ar equipment'.	2° - Push or pull to achieve the opening position.

TILTING HEAD

It is possible to adjust head's panel in different positions:

Relax



Head's Elevation

- 1° - Press the red lever
- 2° - Pull or push the head's panel to achieve the desired position



ADJUSTABLE FEET POSITION

Adjustable feet in three different positions:

Footrest and Knees-Flex Position



Lower Legs part Elevation



Legs Elevation



SECURITY INFORMATION



This equipment can be dangerous:

- Danger of Entrapment.

WARNING: USE WITH CAUTION!



- Danger of Entrapment.

MAINTENANCE



- Perform visual inspection every fifty cycles of use.
- Lubrication bimonthly elements of mechanical operation with high viscosity lubricant.
- Verification of tightness of binding elements quarterly. Make adjustments if necessary.
- Use cleaning products compatible with aluminum, stainless steel and rubber.

ACCESSORIES

Restraints




Shoulder Harness restraints



Legs and Feet restraints

Restraints Instalation

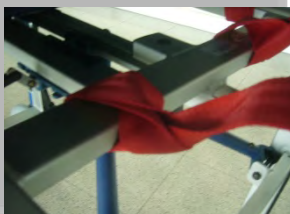

1° - Tying restraint straps in inner bars.



2° - Pass the restraints through the plate slots



3° - Join belt buckles.

ACCESSORIES

IV Pole



IV Pole



IV Pole in use.

Instruments Tray



Instruments Tray



Instruments Tray in use.



Rua de S. Caetano, 551 Apartado 526
4411-701 Canelas; Vila Nova de Gaia, Portugal
Telf: +351 227157100 , Fax: +351 2271244
support@arequipment.com
www.arequipment.com

SERIAL NUMBER

