



**Instruction Manual:
M761 – Monoblock 4 Parts**

M761 – MONOBLOCK 4 PARTS

The Monoblock of 4 parts M761 (a modern M760) is an equipment with a permanent stretcher and it's manually activated. Its main goal is the transportation of patients with reduced mobility and post-surgery. This product is developed and produced by Auto Ribeiro.

Equipment's Overview



Equipment's Side Views





Equipment's Top View






FEATURES

- Approximate Weight:
43 kg
- Maximum load allowed:
250 kg
- Approximate Dimensions (closed):
1915 x 568 x 455 mm
- Approximate Dimensions (open):
M761 1915 x 568 x 990 mm
M761S 1915 x 568 x 930 mm
M761H 1915 x 568 x 1090 mm



FRONT LEG COMMAND (in Intermediate Position)

<u>Safety System</u>	
	1° - Lift the tab (located on the right side of the cart).
	Final Position.



WHEELS

<u>Swivel Wheels</u>	
	2 Swivel Wheels.
<u>Stationary Wheels</u>	
	2 Stationary Wheels.
	<i>Lock of the Stationary Wheels</i> 1° Activate the tabs with your foot so the wheels stay locked.



REAR LEG COMMAND

<u>Safety System</u>	
	1° - Lift the safety tab.
	2° - Press the LEFT lever until the movement begins and keep it pressed. (NOTE: to stop the descent of the rear leg in the intermediate point you must press the lever and release it immediately.)



FRONT LEG COMMAND

<u>Safety System</u>	
	1° - Lift the safety tab.
	2° - Press the RIGHT lever until the movement begins and keep it pressed.

EXTENSIBLE HANDLES

<u>Safety System</u>	
	1° - Press the button with your finger.
	2° - Pull or push to adjust to the desired length.

PROTECTION HANDLE

<u>Safety System</u>	
	1° - Press the button with your hand.
	2° - Pull or push to adjust the open position.

INCLINABLE HEADBOARD

The coverage in the headboard area can be adjusted in different positions:

Rest



Lifting of the head

- 1° - Press the red lever
- 2° - Pull or push the headboard



INCLINABLE FEET

The coverage in the feet area can be adjusted in 3 different positions:

Rest and Bending of the knees



Lifting of the interior part of the Legs



Lifting of the Legs



SAFETY WARNINGS



This equipment can be dangerous:

- Entrapment Danger.

WARNING: USE WITH CARE!



- Entrapment Danger.

MAINTENANCE



- Execute visual inspection every fifty cycles of use.
- Bimonthly lubrication of the mechanical operating elements with lubricant of high viscosity.
- Verify the grip of the connecting elements quarterly. Proceed with adjustment if necessary.

CLEANING AND DISINFECTION

- Execute cleaning and disinfection after every use of the equipment.
- Use a disinfectant that acts against bacteria, fungi and virus.
- Do not store the equipment without drying it completely.
- Do not use chemical products that are corrosive and abrasive.

ACCESSORIES

Fixation Belts




Chest Belt, "like a vest".




Belts for the legs and feet.

Belt's Installation

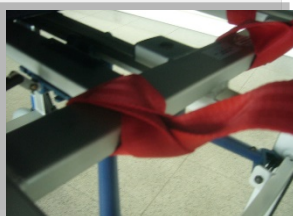

1° - Tie the belts in the interior bars.

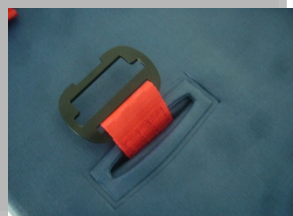



2° - Make the belts pass by the slot of the plates.



3° - Attach the belts buckles.

ACCESSORIES

<u>Serum Support</u>	
	Serum Support at rest.
	Serum Support in use.
<u>Table for Medical Devices</u>	
	Table at rest.
	Table in use.



Rua de S. Caetano, 551 Apartado 526
4411-701 Canelas; Vila Nova de Gaia, Portugal
Telf: +351 227157100 , Fax: +351 2271244
support@arequipment.com
www.arequipment.com

SERIAL NUMBER

