

# User Manual: M7 / M7BB: Monobloc



## **M7 / M7BB: Monobloc**

The Monobloc M7 / M7BB is an equipment with fixed stretcher and manually operated. Its main objective is the transport of patients with mobility difficulties and post- operation condition. This product is developed and produced by Auto Ribeiro Lda..

### **Equipment's Overview**



### **Equipment's Lateral view**



**Top view of Equipment**



**CHARACTERISTICS:**


- Weight: 45 kg
- Maximum load capacity: 250kg
- Monobloc dimensions (closed): 1945 x 568 x 480 mm
- Monobloc dimensions (open): 1945 x 568 x 1010 mm

## WHEELS

### Swivel Wheels



4 Swivel wheels

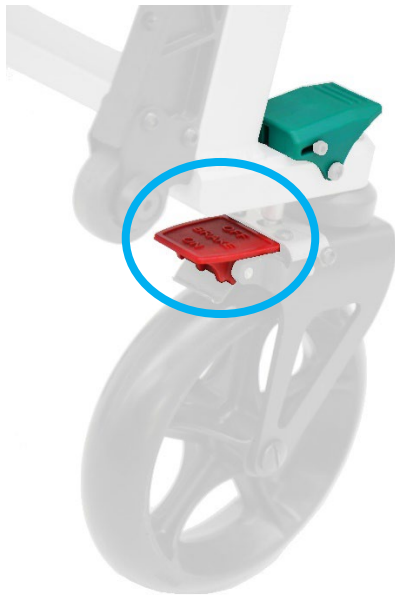

#### **Block of two swivel wheels:**

1° - Engage the tabs with the foot to lock the wheels.

## REAR LEG CONTROL

### Security System



1° - Press the left lever until it moves and keep it pressed.



## FRONT LEG CONTROL

### Security System





1° - Press the right lever until it moves and keep it pressed.

## TELESCOPIC HANDLES

<u>Security System</u>	
	1° - Press button with finger.
	2° - Push or pull to fit the desired length.

## SWING DOWN SIDEARMS

<u>Security System</u>	
	1° - Press red button with hand.
	2° - Push or pull to achieved the desired opening position.


## ADJUSTABLE BACKREST

The backrest can be adjusted in different positions:

### Flat position



- 1° - Press the red lever
- 2° - Push or pull the backrest

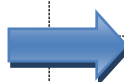

## ADJUSTABLE FEET POSITION

***Footrest Position***







***Legs Elevation***






## SAFETY WARNINGS

	<p> <b>This Equipment can be dangerous:</b></p> <ul style="list-style-type: none"><li>• Risk of entrapment</li></ul> <p><b>ATTENTION: USE WITH CAUTION!</b></p>
	<p> Risk of entrapment</p>

## MAINTENANCE

<p></p> <ul style="list-style-type: none"><li>• Perform visual inspection every fifty cycles of utilization.</li><li>• Lubrication elements of mechanical function every six months with high viscosity lubricant</li><li>• Checking of tightness of binding elements of every three months. Proceed to adjust if necessary</li></ul>
---

## ACCESSORIES

<p><b><u>Fixing Restrain Straps</u></b></p>	
	



Rua de S. Caetano, 551 Apartado 526  
4411-701 Canelas; Vila Nova de Gaia, Portugal  
Telf: +351 227157100 , Fax: +351 2271244  
[comercial@arequipment.com](mailto:comercial@arequipment.com)  
[www.arequipment.com](http://www.arequipment.com)